



# Instruction for use OVIXcell



*Caution: Please read this entire insert before use of the product.*

**OVIXcell** has been developed specifically to improve semen preservation for rams at a temperature of 4C° for 24 hours

## PRODUCT DESCRIPTION

### Composition

- Water
- Salts
- Sugars
- Electrolytes
- Glycerol
- Antibiotics
- Animal free protein

### Technical characteristics:

- Extender ready to use to add to ram sperm manufactured by IMV-Technologies in an ISO 9001 Version 2000 certified plant.
- Light brown colour
- Sterilized by irradiation

### Precaution for use

- Do not use in case of damage to the product or its packaging

## INSTRUCTION FOR USE

- Bring to 34 / 37°C in a water bath
- Add to fresh collected ejaculate to a dilution rate according to the length of preservation  
We advise the following:  
Cervical AI with cooled semen: 300 millions spz/dose for usage 24 hours after collection  
Cervical IA with fresh semen: 250 millions spz/dose
- Cool down to 4 – 5°C for 90 mn
- Fill straws

## PRESERVATION

- Do not use beyond the expiry date indicated on the label
- Store in a dry place at + 2 to + 8 °C. Avoid exposure to direct sunlight

## PRESENTATION

Description	Packaging	Reference
OVIXcell	100 ml x 4 flasks of ready to use extender	020997