

### Connecting the probe

First, check the probe plug and the socket on Duo-Scan are clean and free of dirt/debris. Align the probe connector with scanner socket (the probe has to be rotated to line up the plug and socket features before it will push in place). A clean connector will easily push into place once aligned correctly. Rotate the locking ring to secure the connector.

### Switching on

Connect your viewing device (align the plug and screw the locking ring home). Clip a charged battery onto the scanner. Press and hold the power button until the scanner beeps.

### Changing settings

Pressing any key displays the menu on the viewing device. The menu is displayed for approximately 6 seconds then disappears, pressing any button switches the menu back on. On the menu the four text blocks at the bottom of the display on the viewing device describe the current function of the four buttons on the Duo-Scan front panel.

#### Buttons



Scrolls through preset modes



Switches between power or gain adjustments



Power or gain down



Power or gain up



Short press freezes image. Long press switches on/off.

### Battery meter



A battery icon is displayed on the viewing device whenever the menu is active. Each bar on the battery meter represents approximately 1 hour of capacity. The battery meter will flash when the remaining capacity is 5 minutes.

#### Note:

All IMV Technologies products are to be used for animal applications only.



## Contact us now

North America +1 (507) 529 8200

France +33 (0) 5 45 92 03 57

### Follow us

[www.imv-technologies.com](http://www.imv-technologies.com)

