


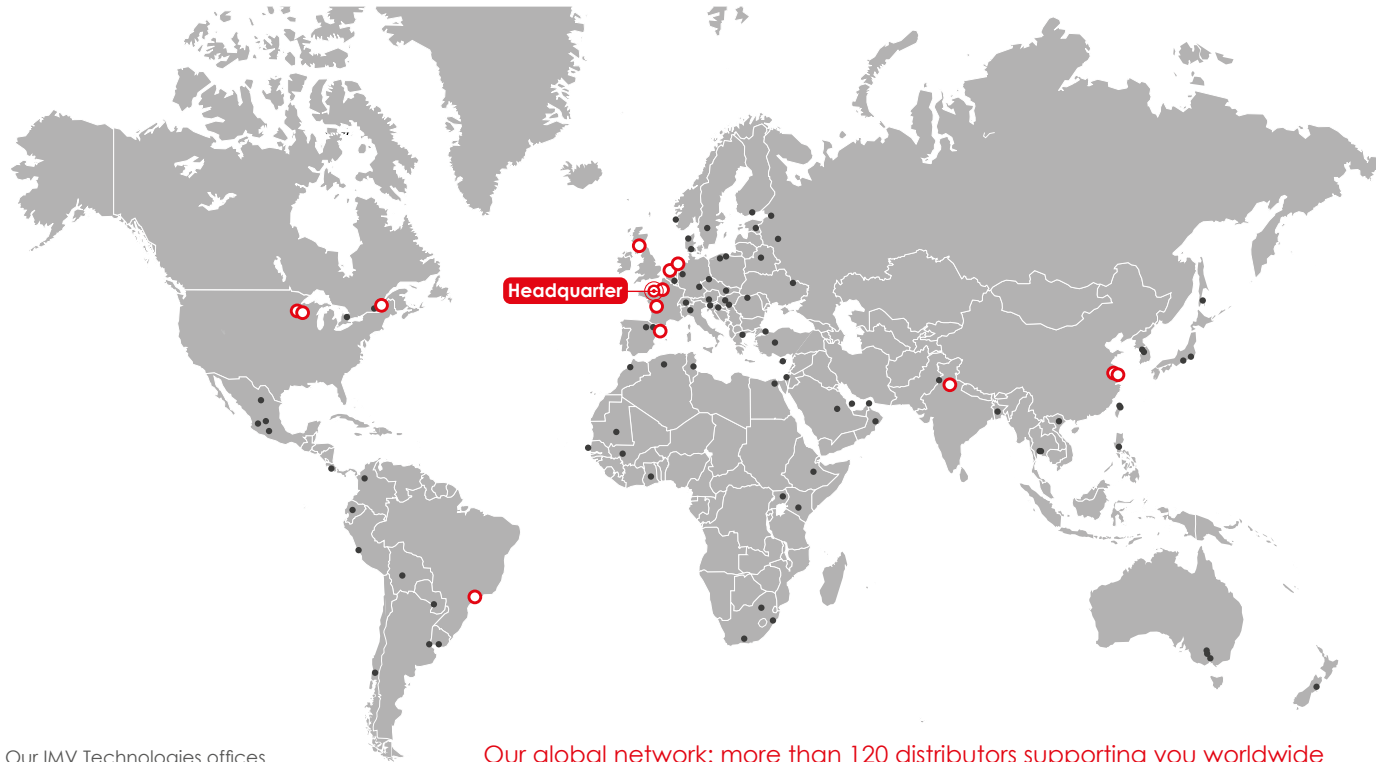

Our global network: more than 120 distributors supporting you worldwide

- Our IMV Technologies offices
- Main distributors

#### France

IMV Technologies Headquarters  
+33 2 33 34 64 64  
61300 - L'Aigle  
welcome@imv-technologies.com

#### IMV imaging

16000 - Angoulême  
france@imv-technologies.com

#### Polysem

78120 - Rambouillet  
welcome@imv-technologies.com

#### The Netherlands

Leja  
Nieuw-Vennep - 2153 GN - North Holland  
info@leja.nl

#### NIFA Technologies

Leeuwarden - 8938 AS - Friesland  
www.nifa.nl

#### Scotland

IMV Technologies UK  
Bellshill - ML4 3NJ - North Lanarkshire

#### Spain

Imporvet  
Barcelona - 08540 Centelles - Catalonia  
int@import-vet.com

#### United States

IMV Technologies USA  
Brooklyn Park - 55445 - Minnesota  
contact@imvusa.com

#### IMV Technologies USA

Rochester - 55901 - Minnesota  
info@imvusa.com

#### Canada

Ro-main  
Saint-Lambert-de-Lauzon  
G0S 2W0 - Quebec  
info@ro-main.com

#### Brazil

IMV Technologies Brazil  
Campinas - 13030-300 - Sao Paulo  
contato@imvbrasil.com.br

#### China

IMV Technologies China  
201203 - Shanghai  
contact@imvchina.com

#### Nelson Techno Medical Co

Changzhou - Jiangsu - China  
contact@imvchina.com

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
#### India

IMV Technologies India  
Gurugram - 122016, Haryana  
contact@imvindia.com



# Kineo Start

COMPACT SEMEN ANALYZER  
**DESIGNED FOR SIMPLICITY**



www.imv-technologies.com

Ref: 2026/01/05/A\_EN



# Kineo Start

Kineo Start is a compact system for semen analysis; a versatile solution **for both semen dose production and quality control**: Kineo Start supports processing decisions during **production** and **ensures dose quality before insemination**.

**Species:**

- Swine
- Equine
- Bovine
- Ovine
- Caprine



Compact semen analyzer



Robust and transportable

## Simple process



- AI-powered imaging innovation
- Focus-free imaging
- Auto-light adjustment
- Manual heated stage

### Results:

- Concentration
- Motility
- Progressive

### Calculation:

- Number of doses
- Extender volume

### Data:

- Statistics
- Sire database



Step 1: Sample dilution.



Step 2: Kineo slide filling on a heated stage.



Step 3: Semen analysis with Kineo Start.

## Designed for simplicity


1. Intuitive software designed with input from end users across multiple countries, covering different profiles such as vets, farmers, and lab technicians.
2. Easy-to-use hardware, no adjustments required.
3. Easy training with step by step tutorial and software tips.



## Trust your results


1. New technology designed with 50+ years of technical expertise and using industry-wide approved thresholds.
2. Amplified by AI developed with top research experts, producing reliable results.
3. Results approved by Spallanzani Institute and Zurich university.

CCC: Concentration Kineo Start vs Concentration Nucleocounter.  
 $y = 0.974x$  |  $R^2 = 0.992$  | CCC = 0.974 | n = 88

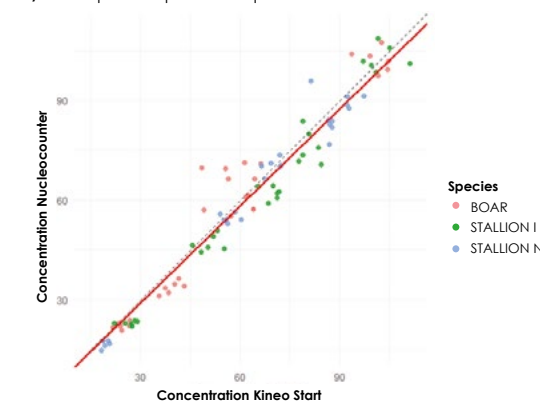

## Prove your semen doses quality and optimize your performance

1. Customizable official reports with your logo to include with semen shipments and prove quality.
2. Track and compare male performance to optimize animal management and improve semen quality.
3. Safe and easily accessible data whenever needed.

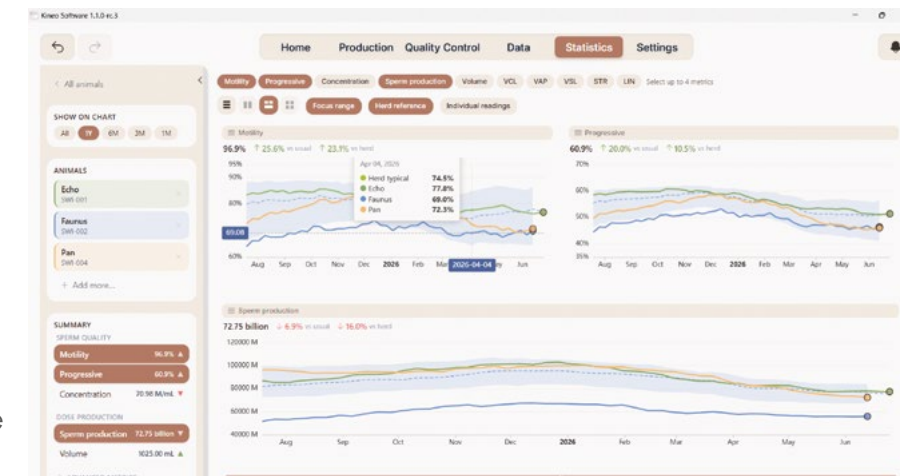

## Hardware specifications

**Dimensions:** 110W x 100D x 240H mm

**Weight:** 1.47 kg

**Heated Stage**

Temperature Control: 37°C +/- 0,5°C

**Languages**

French, English, Italian, Chinese, Spanish, Russian, Portuguese, Japanese, German, Vietnamese

**Compatible consumables**

Type: Kineo slide 5 analysis (Patented)

### Minimum Computer Configuration Requirements:

- Intel® Core™ i5-1335U or Celeron J6412 @ 2.60GHz
- Intel® Iris® Xe graphics or higher
- 8 GB DDR4 memory
- 500 GB M.2 SSD, PCIe NVMe (1 PC)
- Windows 10 or later
- Minimum 1 USB Type-C port compatible with USB 3.0