

Connected to
... SEMEN COLLECTION

NEW

MaxCollect

Collection cone for semi-automated collection and Collectis.

PRODUCE MORE DOSES EASILY



MaxCollect

Efficiency

Collect up to 10% more doses of semen with MaxCollect.

No more wasting semen

- Optimized filtration system:
 - More than 400 holes
 - No clogging (tapioca, semen)
- Fast filtration rate: only a few seconds are needed to filter.


Faster filtration of large ejaculates

Large ejaculates are filtered quickly without blocking the flow of the semen through the cone thanks to:

- Higher filtration volume
- New shape of the cone with larger filtration capacity
- No risk of obstruction from gel

Priority on quality

In synergy with Collectis: bacterial load divided by 10

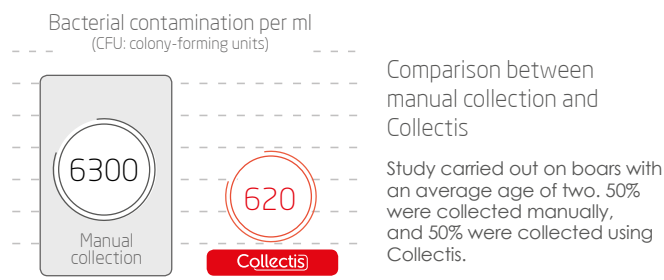

With MaxCollect, the bacteria management is the same.

Improved working conditions


MaxCollect is stiffer thanks to a new design and material. This makes it easier to handle and connect to the artificial vagina.

Effect of the cone on the number of theoretical doses.

Number of theoretical doses	Mean \pm sd
Control (n = 378)	39.4 \pm 12.6
New (n = 287)	43.2 \pm 12.9
P-value	0.0018

n: number of ejaculates

Protocol: test done with Collectis in a boar stud in France with 92 boars (age: 7 months to 4 years old).

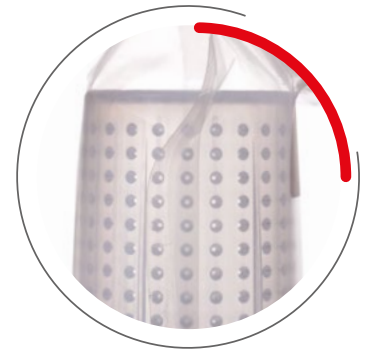

+3,8
DOSES

Easier visual control

Thanks to a better overall view of the collection, it is now easier to check for impurities or for blood in the ejaculate.



New shape for
easy handling